

Please read the information contained herein carefully.

**FreshKon®**

**C L E A R**

Comfort and Clarity

**Multi-Purpose  
Solution**

### FreshKon® CLEAR Multi-Purpose Solution

FreshKon® CLEAR Multi-Purpose Solution (MPS) cleans, rinses, removes protein, disinfects, lubricates and stores all soft (hydrophilic) contact lenses including cosmetic and Silicone Hydrogel contact lenses.

FreshKon® CLEAR is a Multi-Purpose Solution that has ionic molecules which combine with protein's charged molecules and enhance the release of protein deposits from the lens. HPMC forms a lubricating shield on the lens, keeps in moisture and lubricates the lens to give long lasting comfort to your eyes.

#### Warnings

It is important to follow the instructions of your eye-care practitioner and the directions as indicated on the package and label for proper handling and lens care. Failure to do so may result in severe eye infection or loss of vision. If you experience any eye discomfort, irritation, pain and redness or blurred vision, remove the lenses and consult your eye-care practitioner IMMEDIATELY.

#### **DO NOT MIX WITH OTHER FLUIDS EXCEPT AS DIRECTED.**

Do not use concurrently with any eye medication except on medical advice.

#### Adverse Reaction

The following problems may occur: Stinging or burning sensation in the eyes, irritation, discomfort, foreign body sensation, excessive tearing, redness, blurred vision, halos or ghosting around images, sensitivity to light or dry eyes.

If you experience any of the above, remove the lenses and consult your eye-care practitioner IMMEDIATELY.

#### Contraindication

If you are allergic to any ingredients in FreshKon® CLEAR MPS, do not use this solution.

#### Precautions

- Do not purchase or use FreshKon® CLEAR MPS if bottle is opened or tempered/damaged.
- Do not reuse FreshKon® CLEAR MPS.
- Do not use with any heat (thermal) disinfecting method.
- To avoid contamination, do not touch the dropper tip of the container to any surface.
- Keep the container tightly capped when not in use.
- Store at room temperature (15°C - 30°C) and away from direct sunlight.
- Discard any remaining solution 90 days after first opening the bottle.
- Use before the expiry date.
- Keep out of reach of children.
- Do not freeze.

#### Instructions

- Always wash your hands thoroughly and dry (lint-free method) before handling your contact lenses.
- Apply a few drops of FreshKon® CLEAR MPS on the lens and gently rub for 20 seconds.
- Rinse the lenses thoroughly with FreshKon® CLEAR MPS and soak the lenses for at least 6 hours before use.
- Rinse soaked lenses with FreshKon® CLEAR MPS before placing into eyes.
- After use, always empty the lens case, clean thoroughly and air dry the lens case, including the covers.

#### Recommendation

Change your lens case and accessories at least once every 3 months.