

Instruction For Use

FreshKon®

CLEAR

Refreshing Drops

FreshKon® CLEAR Refreshing lubricate and moisturize your contact lenses, thus soothing and refreshing your eyes for long-lasting comfortable lens wear.

Suitable for all soft contact lenses including cosmetic and silicone hydrogel lenses.

Contents

A sterile, isotonic solution containing Boric Acid, Borax, Sodium Chloride, Potassium Chloride, Edetate Disodium (EDTA), Polyhexanide (PHMB), Hypromellose (HPMC) and Poloxamer.

Warnings

If you experience any eye discomfort, irritation, pain and redness or blurred vision, remove the lenses and consult your eye-care practitioner IMMEDIATELY.

DO NOT MIX WITH OTHER FLUIDS EXCEPT AS DIRECTED.

Do not use concurrently with any eye medication except on medical advice.

Adverse Reaction

The following problems may occur: Stinging or burning sensation in the eyes, irritation, discomfort, foreign body sensation, excessive tearing, redness, blurred vision, halos or ghosting around images, sensitivity to light or dry eyes.

If you experience any of the above, remove the lenses and consult your eye-care practitioner IMMEDIATELY.

Contraindication

Do not use if you are allergic to any ingredients in FreshKon® CLEAR Refreshing Drop.

Precautions

- To avoid contamination, do not touch the dropper tip of the bottle to any surface.
- Once opened, discard any unused solution after 30 days.
- Store at room temperature (15°C - 30°C) and away from direct sunlight.
- Do not use FreshKon® CLEAR Refreshing Drop if safety ring on bottle is broken or missing.
- Keep bottle tightly closed when not in use. Use before the expiry date.
- Follow the daily lens care procedures as recommended by your eye care practitioner.
- Keep out of reach of children.

Instructions

- Twist the cap to break open the safety ring.
- Apply 2 – 3 drops to each eye whenever needed. Blink several times.
- Screw the cap tightly after use.
- It is advisable to consult an eye care practitioner before changing your lens care system.

STERILE A
Sterilized by filtration

Brand Owner
Oculus Private Limited
10 Kaki Bukit Avenue 1 #06-08
Singapore 417942
www.freshkon.com

Last revision July 2020

SFCRFD-IFUK(0)